SPORT-ELEC®

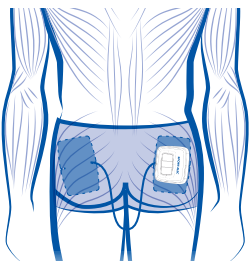
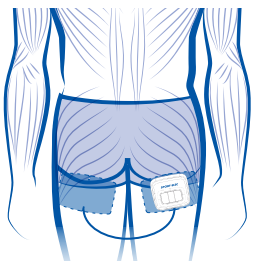
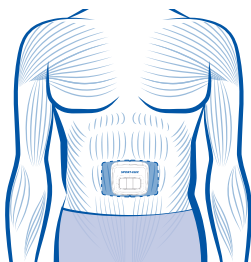
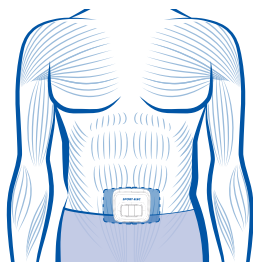
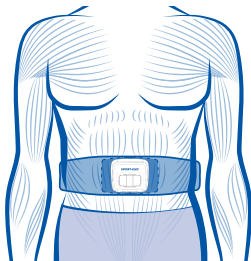
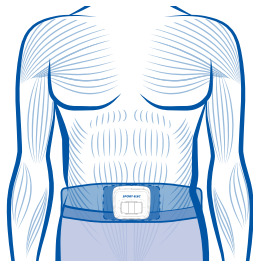
ULTRANOMADE®


Mode d'emploi

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X 3
AAA LR3 1,5 V
Non fournies

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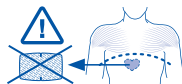I. USING THE DEVICE WITH THE ELECTRODE

This function allows you to use the adhesive electrode by connecting it directly to the unit.

1 Getting started

- ▶ Remove the plastic backing sheet from the electrode.
- ▶ Moisten it by spreading a few drops of water onto the adhesive gel using your fingertips and wait 2 minutes.

2 Positioning of the electrode

- ▶ Position the electrode on your body (cf. positions shown for guidance only).
- ▶ Fix the electrode to the unit using the snap fasteners taking care to place the belt (chapter II).

Note: The electrode are for personal use. For hygiene reasons, they must always be used by the same person.

3 Starting the program

Start your device by following the "Starting the device" procedure (chapter III).

Contractions should be firm but must never cause any pain.

If the electrode cause you any discomfort: reduce the power level, move the electrode and then gradually increase the power. Also ensure that the electrode are sufficiently moistened.

Now that the settings have been optimized, let your device get to work for the full duration of the program.

4 End of program

- ▶ Once the program finished, switch off the device by pressing the "⏻" button, then disconnect the unit.
- ▶ Remove the belt.
- ▶ Gently remove the electrode from your skin.
- ▶ Moisten it slightly with a few drops of water and replace it carefully onto its transparent plastic backing sheet. Keep the adhesive electrode in a plastic bag in a cool place (for example, a separate compartment in your fridge).

II. USING THE DEVICE WITH THE BELT

The UltraNomade belt has 2 locations where the wireless electrode specially designed by SPORT-ELEC® is clipped.

Warning! Only use electrodes supplied by SPORT-ELEC®.

To obtain new electrodes, contact your usual retailer or order them from our website www.sport-elec.com.

1 Connection and positioning of the belt

- ▶ Check that your device is switched off.
- ▶ Open out the belt.
- ▶ On the back side of the belt, place the electrode with snaps across the belt holes.
- ▶ On the front side, place the UltraNomade device and connect it to electrode trough the belt thanks to the snaps.
- ▶ Place the belt around the waist and adjust it using the Velcro strip, so that there is a good contact between the electrodes and the skin.

If you experience any tingling, switch off the device and check that the electrode is in contact with the skin. If the problem persists, contact the after-sales service: sav@sport-elec.com

2 Starting the program with the belt

Once your device is connected, make yourself comfortable and switch it on following the "Starting the device" procedure detailed in chapter III.

You can carry out your normal activities while using UltraNomade.

3 End of program

- ▶ Once the program has finished, switch off the device and unplug it from the belt.
- ▶ Remove the electrode.
- ▶ Moisten it slightly with a few drops of water before replacing it onto the transparent plastic backing sheet.

III. STARTING THE DEVICE

This electro-stimulation device is a state-of-the-art appliance; it complies with the standards in force in the field of medical application.

UltraNomade can be used directly with electrodes or with the electro-stimulation belt onto which an electrode and the unit are attached.

The "man machine" interface has been made as simple as possible.

The number displayed on the screen (from 1 to 4) indicates the program selected via the next "P" key.

The graduated scale located to the screen enable you to see the power level of the unit.

You will also find 4 control keys:

- ▶ 1 "⊕" key (red icon) - press briefly.
- ▶ 1 "P" key to select the program - press briefly.
- ▶ 1 pair of "+/-" keys to adjust the stimulation power of each channel.

Note: A long press (1 second) on the "-" key brings the power quickly back to zero.

1 On/Off

- ▶ A brief press on the "⊕" key turns on the device.
Each time it is switched on, the device carries out an auto-test on 11 safety parameters.
During the auto-test, the screen lights up for ½ second.
- ▶ If an internal defect is detected, all program numbers flash.
In this case, switch off your device and send it back to your retailer's after-sales service.
- ▶ A long press (of more than 3 seconds) on the "⊕" key activates the auto-test.
Note: It is possible to stop the ongoing auto-test by switching off the device (press the "⊕" key).

2 "P" Program selection

- ▶ Programs are shown on the right side of the screen by the numbers from 1 to 4.
- ▶ Simply press the "P" button briefly one or several times.
Program numbers are displayed in sequence from 1 to 4 on the screen.
As soon as a program is selected, the corresponding number is displayed.
- ▶ Once the number of the required program is displayed, start your session by pressing the "+" key on the unit.

3 Program

- ▶ The program number remains lit on the screen for its full duration.
- ▶ When a program is running, the "P" key is inactive.
- ▶ However, if you have started a session and wish to change the program, you must turn off the device and then switch it back on in order to select another program.
- ▶ There are 64 levels available to regulate the power from 0 to 64.
 - Segments from 0 to 8: there are 6 power levels, or 6 successive presses on the "+" key in order to display the following segment.
 - Segments from 8 to 10: there are 7 power levels, or 7 successive presses on the "+" key in order to display the following segment.

4 End of the program - Stop during the program

- ▶ At the end of the program, the device stops automatically.
- ▶ To stop the device during the program, bring the power level back down to zero ("–" key), or press the "⏻" key.
- ▶ Emergency stop: press the "⏻" key.

5 Disconnection

Your device is equipped with an electrode disconnection detector.
If there is no contact between the electrode and the human body, the flow of current is interrupted.
The first bar of the bar graph flashes to warn you of a disconnection.

6 Checking the battery levels

Battery level low: bars n° 6 on each bar graph flash.

Battery level critical: bars n° 10 on each bar graph flash, then the device turns itself off.

Replace old batteries. Always use high quality batteries. The use of bad quality batteries can cause this unit to malfunction.

IV. 4 PROGRAMS

1 FIRMNESS (20 min)

Pulse rate = 70 Hz
ON time = 4.5 secondes

3 MUSCLE FORCE (30 min)

Pulse rate = 50 Hz
ON time = 6.5 secondes

2 MUSCLE CARE (26 min)

Pulse rate = 60 Hz
ON time = 5.5 secondes

4 MASSAGE (40 min)

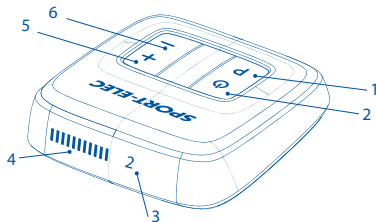
Pulse rate = 50 Hz
ON time = 7.5 secondes

V. DESCRIPTION

1 Contents

- ▶ 1 UltraNomade unit
- ▶ 1 belt
- ▶ 1 wireless adhesive electrode pad large size
- ▶ 1 pair of linked adhesive electrode pads
- ▶ This user manual

2 Description du boîtier

1. Program selection "P" key.
2. "⏻" key ON/OFF / emergency stop key.
3. Program indicator.
4. Power level indicator.
5. Increase power key (level 0 to 64).
6. Decrease power key (level from 64 to 0).

3 Insertion of batteries

Check that the device is switched off.

Open the batteries compartment, insert three 1,5 V AAA LR03 alkaline batteries into the compartment then close the case. Comply with “+” and “-” indications.

Do not mix used and good batteries. For an optimal use of your device, it is advised to use good quality batteries.

Automatic stop function: The device turns itself off after 5 minutes of inactivity if:

- ▶ No keys are pressed.
- ▶ No impulse is delivered.

Battery level low: bar n°6 on the bar graph flash.

Battery level critical: bar n° 10 on the bar graph flash, then the devices turns itself off.

4 Quick function test

Test the device on your biceps:

- ▶ Connect an adhesive electrode to your unit.
- ▶ Test your device on program 2.
- ▶ Gradually increase the power.
- ▶ You will begin to feel contractions, the device is working.

VI. IMPORTANT INFORMATION

1 Please read this before using the device

You have just acquired an effective and powerful device.

You must take time to discover how it works.

- 1 - Read the instructions carefully
- 2 - Ensure that the batteries are well installed prior to use
- 3 - Test the device on your biceps following the quick test procedure above.

2 Precautions

- ▶ Hypersensitivity to electro-stimulation may cause a skin reaction in certain individuals.
- ▶ Keep this device out of reach of children or mentally-impaired persons. This product contains small parts which can be easily swallowed.
- ▶ Only use UltraNomade with accessories supplied and recommended by SPORT-ELEC® INSTITUT.
- ▶ Do not use UltraNomade when driving, operating machinery, or during any activity where electro-stimulation could result in the risk of injury to the user.
- ▶ Always position the belt in accordance with the information provided in this manual.
- ▶ Do not apply the electrodes to the lower part of your neck. Muscle contraction may be very strong and cause respiratory problems, as well as affect heart rate and blood pressure.
- ▶ Do not apply to the heart area, as electro-stimulation may cause heart rhythm disturbances.
- ▶ Do not place electrodes on the face, head or temples. The effects of electro-stimulation on the brain are unknown.
- ▶ Only apply the electrodes to clean and healthy skin. Do not carry out electro-stimulation on an open wound or on swollen, infected or inflamed areas, or on skin rashes (e.g. phlebitis, thrombophlebitis, varicose veins...).
- ▶ Do not use in the bath, shower or for any activity in contact with water.
- ▶ Do not use when sleeping.
- ▶ Do not use if you are fitted with an IUD contraceptive device, as contraceptive efficiency under electro-stimulation has not been established.
- ▶ Do not use on parts of the body other than those for which the accessories have been designed.
- ▶ The electrodes are for personal use. For hygiene reasons, they must always be used by the same person.
- ▶ Do not use on animals.

Special precautions should be taken in the following circumstances:

- ▶ Persons with suspected epilepsy.
- ▶ If you have a tendency to internal bleeding or bleeding following an injury or fracture.
- ▶ Following a surgical operation without medical advice (electro-stimulation may disrupt the recovery procedure).
- ▶ In areas of the skin where sensitivity is impaired.
- ▶ Do not use if you are fitted with an IUD contraceptive device, as contraceptive efficiency under electro-stimulation has not been established.
- ▶ Do not use on parts of the body other than those for which the accessories have been designed.
- ▶ The electrodes are for personal use. For hygiene reasons, they must always be used by the same person.
- ▶ Recycle or dispose of the battery in accordance with all federal, state and local laws. To avoid fire and explosion hazard, do not burn or incinerate the battery.
- ▶ The device is not suitable for use in the presence of flammable anesthetic mixtures with air or with oxygen or nitrous oxide.
- ▶ The device requires no calibration.
- ▶ The user must check that the equipment functions safely and see that it is in proper working condition before being used.
- ▶ Keep the equipment away from children and pets.

THERAPEUTIC CLAIMS

- ▶ **Individuals:** for the strengthening and/or maintenance of muscle strength, to remodel your figure and for pain relief.
- ▶ **Physiotherapists:** for the relief of muscular tension and within the scope of functional rehabilitation.
- ▶ **Hospital sector:** for the prevention of muscular amyotrophy and for pain treatment.

ADVERSE EFFECTS

- ▶ Skin irritation.
- ▶ Mild burns the skin where the electrode is applied.

Seek the advice of your doctor in the following cases:

- ▶ You suffer from serious back pain.
- ▶ You have a serious illness not mentioned in this manual.
- ▶ You have recently undergone a surgical procedure.
- ▶ You take insulin for diabetes.

RECOMMENDATIONS FOR USE

- ▶ You can restart a program cycle on another muscle zone.
- ▶ For optimum use, refer to the diagrams shown in this manual.
- ▶ The units can be cleaned using a cloth dampened with a disinfectant solution.

CONTACT SPORT-ELEC® INSTITUT OR AN AUTHORIZED DISTRIBUTOR IF:

- ▶ You experience any irritation, skin reaction, hypersensitivity or any other reaction. However, please note that any redness of the skin on contact with the electrode is normal and disappears shortly after.
- ▶ Correct use of the electro-stimulation belt should not cause any discomfort. SPORT-ELEC® INSTITUT declines all responsibility if the user does not follow the instructions described in this manual.
- ▶ If for any reason you are in any doubt concerning the use of this device, consult your doctor prior to use.

RESTRICTIONS OF USE

- ▶ The belt must not be connected to another device or any other object other than the UltraNomade.
- ▶ The complete device must be used, with all detachable parts in place.
- ▶ Do not use the electro-stimulation unit when driving or operating machinery.
- ▶ Do not use in damp conditions.
- ▶ Use the device in a clean environment (dust and dirt-free...).
- ▶ Keep the device away from all sources of heat.
- ▶ Keep the device away from any wave-emitting or high frequency equipment.
- ▶ To reposition the electro-stimulation unit and the electrode during the session: stop the program in progress, reposition and restart once the belt has been readjusted.

REPAIRS

If your device is not working correctly, stop using it and contact your retailer.

VII. WARNINGS

DO NOT USE THIS DEVICE SPORT-ELEC®

- ▶ If you carry a pacemaker.
- ▶ If you are pregnant.
- ▶ During first 6 weeks after baby birth.
- ▶ In case of neuromuscular diseases.
- ▶ In case of haemorrhagic diseases.

ASK FOR ADVICE FROM YOUR PHYSICIAN

- ▶ If you have electronic or metallic implants (pins...).
- ▶ In case of skin condition (wounds...).

DO NOT APPLY ELECTRODE

- ▶ On the face, eyes, mouth, or on the front of the neck.
- ▶ On one side to the other of (or through) the head.

POTENTIAL DANGERS

- ▶ The simultaneous connection of a patient to a high frequency surgical unit may cause burns at the points of contact of the stimulator's electrodes, and the stimulator may be damaged.
- ▶ Operation in close proximity of a short wave therapy unit may cause instability in the stimulator's output power.

MODIFYING THE UNIT IS FORBIDDEN

- ▶ It is forbidden to modify the unit without the manufacturer's authorization.
- ▶ If the unit is altered, a function check and an appropriate test must be carried out to ensure that it can still be used safely.

VIII. CLEANING AND MAINTENANCE

1 Unit

The unit can be cleaned with a cloth soaked in a disinfectant solution.

2 Adhesive electrode

Rehydrate the adhesive side of the electrode before and after use. A disinfectant agent can be used. They should be stored on the original backing sheet in a plastic bag between 5°C and 10°C (for example in a separate compartment in your fridge).

IX. TECHNICAL CHARACTERISTICS

1 Belt

| | |
|-----------------------------------|----------------------------|
| Reference | EXBELT |
| Size (unisex) | From 34 to 60 |
| Length of the unfolded belt | 126 cm |
| Fabric | 94% polyester, 6% elastane |

2 Adhesive electrode

Class I medical device (Directive 93/42/CEE)

Type BF applied parts.

Adhesive, hydrophilic and hypoallergenic electrodes

References BIGPAD ou EAEXP

Contact area 160x103 mm (BIGPAD) ou 100x50 mm (x 2 - EAEXP)

Estimated life expectancy 40 uses

Please order new UltraNomade-compatible adhesive electrodes on www.sport-elec.com

3 Unit

| | |
|--|--|
| Class IIa medical device in accordance with Directive 93/42/CEE. | |
| Software version | 1 |
| Type of current | Symmetrical biphasic rectangular |
| Number of programs | 4 programs |
| Number of adjustable independent channels | 1 |
| Current | 1 μ l from 0 to 60 mA - from 0 to 60 Vcc - from 0 to 1000 Ω |
| Frequency range/Pulse width range | From 50 to 70 Hz / 234 μ s |
| Power | 3 batteries AAA - LR3 1.5V |
| Unit dimensions and weight | 92 x 81 x 22 mm - 90 g |
| Automatic stop at the end of each program | Yes - after approx. 5 min of inactivity |
| Activation safety | 100% |
| Detection absence of contact | Yes |
| Indicator of power | LCD display: 1 bar graph |
| Indicator of low and/or defective battery | Yes - LCD display |
| Indicator of programs | LCD display: icons 1-2-3-4 |
| Temperature/Relative humidity for storage | 0°C to +40°C / 10% to 90% |
| Temperature/Relative humidity for use | +5°C to +40°C / 20% to 65% |
| Connection of the electrode | 2 snap fasteners |
| Warning |  Read the instructions carefully. |
| Conformity | EN60601-1 / EN60601-2-10 |
| Guarantee/Life cycle..... | 2 years |

4 Standardized symbols / pictograms

Approved medical devices.

0459 (directive 93/42/CEE)

CE marking concerning medical appliances.

Device in contact with the patient, excluding the heart area.

RoHS

These units are free of materials likely to have a negative impact on the environment.

IP22

Protected against the ingress of foreign object \geq 12.5 mm & against dripping water when tilted up to 15°.

Caution :
Read the instructions for use.

Maximum intensity per channel up to 1 k Ω : 90 mA.

Relative humidity for storage :
from 10 to 90%.

Temperature for storage :
from 0 to 40°C.

Name of the manufacturer bringing the device to the market.

Please contribute to protect the environment by observing the disposal instructions.

The manufacturer respects the environmental law and contributes to the cost of recovery and recycling.

CFollow the user manual instructions.

Do not use if you carry a pacemaker.

Do not use if you are pregnant.

This device is intended for domestic use.

Double insulated electrical appliance (> 50V).

X. RECYCLING

Please respect national regulations when you dispose of your product when it reaches the end of its life.
Please take it to a suitable place in order to ensure that it is disposed of in a safe and environmentally-friendly manner.

XI. GUARANTEE

- ▶ We provide a 24 month guarantee for products supplied by us, from the date of purchase.
- ▶ The belt and adhesive electrodes are for personal use.
For hygiene reasons, they can therefore neither be returned nor exchanged.
The life expectancy of the adhesive electrode is estimated at approximately 40 uses.
- ▶ Never attempt to repair or modify the device yourself, or by a repairman not authorized by UltraNomade.
The guarantee becomes null and void if repairs have been carried out by non-authorized persons or if the replacement parts used are not from the same manufacturer.
- ▶ Similarly, only use UltraNomade-compatible accessories manufactured by SPORT-ELEC®.
- ▶ During the guarantee period, in normal use conditions (described in this manual), the defective parts accessories or the entire device will be replaced.
- ▶ Any damage resulting from improper use (knocks...) does not fall under the guarantee; normal wear and tear does not affect the proper functioning or value of the device.
- ▶ The guarantee is only valid if the date of purchase and the stamp and signature of the retailer appear on the guarantee certificate, or upon presentation of the till receipt (see "guarantee certificate" at the end of this manual).
- ▶ No other claims fall under the guarantee, unless otherwise provided by law.

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