SPORT-ELEC[®]

I N S T I T U T

FR

EN

DE

ES

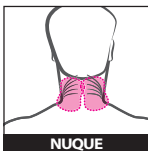
IT

NL


Mode d'emploi / user manual
Manual de uso / Manuale d'uso
Gebrauchsanweisung


FARMADOLOR


NUQUE

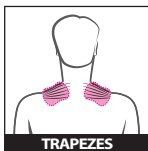
TRAPEZES

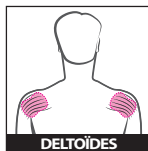
DELTOÏDES

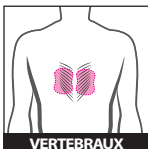
VERTEBRAUX

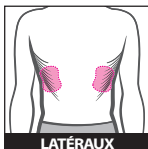
LATÉRAUX

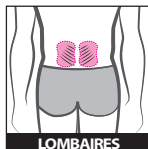
LOMBAIRES

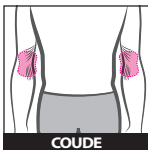
COUDE

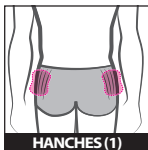
HANCHES (1)

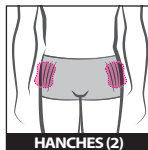
HANCHES (2)

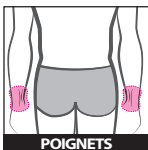
POIGNETS

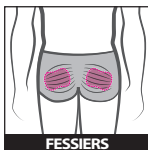FESSIERS

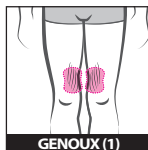GENOUX (1)

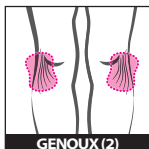GENOUX (2)

CHEVILLES

RoHS  

Made in P.R.C.

FARMADOLOR

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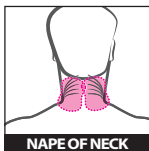
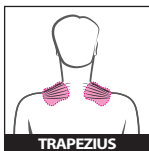
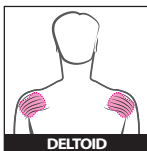
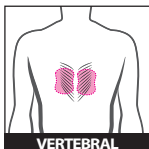
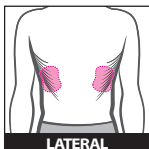
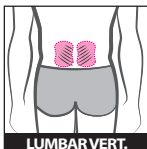
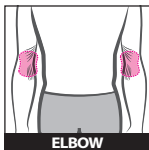
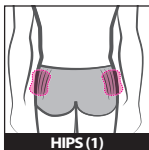
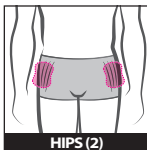
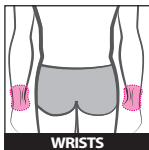
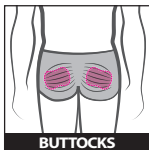
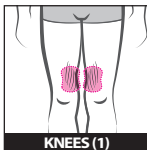
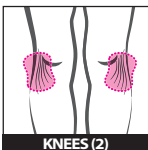
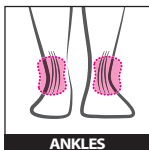
Made in P.R.C.

FARMADOLOR

BEFORE USE: READ THIS USER MANUAL ENTIRELY.

CONTRAINDICATIONS

Do not use the Farmadolor device:

- ▶ On the heart area, on the neck, on the head (face and skull), on the spine.
- ▶ Do not use the power 3 (max.) on the neck, calves, shoulders, forearms.
- ▶ If you're holder: of a pacemaker or if you suffer from any other heart diseases; abdominal implant; of a coil.
- ▶ In case of: cancer, pregnancy, neuromuscular disease, hemorrhagic disease, epilepsy.
- ▶ After delivery, wait for 6 weeks before use.
- ▶ After surgery, wait at least 3 months and consult your physician.
- ▶ On an open wound, swollen or infected places or with inflammation, if rash or itching occur (for example: phlebitis, thrombophlebitis, varicose veins, etc...).
- ▶ Keep away from children or people with mental disabilities. The device contains small pieces that can be swallowed.

RESTRICTIONS OF USE

- ▶ ONLY apply Farmadolor on a healthy and cleaned skin.
- ▶ Do not use in wet area or in the bath, under the shower or for all activities involving contact with water.
- ▶ Use this device in a clean place (without dust nor dirt..).
- ▶ Keep the device away from all heat sources.
- ▶ Keep the device away from all equipment which transmits waves or uses high frequencies.
- ▶ Do not use Farmadolor while driving or using other machines.
- ▶ Do not use while sleeping.
- ▶ Do not touch the inside of the electrode while

Farmadolor is working.

- ▶ Stop using this device immediately if you feel irritation, skin reactions, hypersensitive or any other reaction. Nevertheless, redness of the skin in contact with the electrodes is normal and disappears a few times after use.
- ▶ SPORT-ELEC® disclaims all responsibility if the instructions of this manual are not followed by the user.
- ▶ If any doubt on the use of the device, consult your physician before use.

TROUBLE SHOOTING

If your device doesn't work properly, do not use it and contact your retailer.

SAFETY DURING THE USE

- ▶ The electrode must be coated with your finger with some water before to be placed on the body.
- ▶ The device must be perfectly connected to the 2 generator's wires.
- ▶ It is possible to start again the program cycle on another muscular area.
- ▶ The device must be used entirely, all the removable parts in place (in particular the batteries cover).
- ▶ If the device detects an error while starting, all the three (3) indicator light blink. In this case, change your batteries. If the problem persists call your retailer.
- ▶ Farmadolor requires special caution regarding EMC and must be installed and set up according to EMC information provided in this document. Portable and mobile RF devices communications can affect medical

FR

EN

DE

ES

IT

NL

electrical equipment. This device should not be used adjacent to or stacked with other equipment with them.

ATTENTION: the generator has a security system to avoid improper handling of the device. If the electrode is no longer in contact with the skin, you have 2 minutes to put the electrode back on correctly otherwise the device will stop (safety procedure). If you don't go over 2 minutes, the program keeps going normally.

BATTERY INSERTION

Check if the device is on the « off » position. Open the battery cover at the back of the Farmadolor, insert 2 button cells CR2450 in the slot then close the cover observing the proper polarity (+/-).

Never mix old and new batteries. Do not try to operate the device with other power source. For an optimum use of your device, it is recommended to use good quality batteries.

Low batteries: the three {3} indicator {1, 2 and 3} light steadily.

USE OF THE FARMADOLOR

Your Farmadolor is at the cutting-edge technology, it respects the existing standards in special medical application.

The user interface has been made as simple as possible in limiting itself to two buttons: one to start, the other to adjust the power.

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One short press on the key "⏻" and the device turns on. A second press on the key "⏻" and the device turns off.

2. SELECTING THE POWER "P"

The selection of the power level is indicated

by the switching of the indicator light located above the key "P".

By pressing on the key "P", the indicator light turns off accordingly:

- ▶ **No press: minimum intensity**
(The indicator light n°1 turns on)
- ▶ **1 press: medium intensity**
(The indicator light n°2 turns on)
- ▶ **2 successively press: maximum intensity**
(The indicator light n°3 turns on)
- ▶ **3 successively press: back to minimum intensity** (The indicator light n°1 turns on)

Do not use the power 3 (max.) on the neck, calves, shoulders, forearms.

Note: pause for a few seconds the camera takes place during the program each frequency change.

3. END OF THE PROGRAM / STOPPING

- ▶ The device automatically stops after approximately 60 minutes.
- ▶ Emergency stop: press the start/stop key "⏻".

4. DISCONNECTION

Your device has electrodes disconnection detector with the body. It stops by itself after 4 minutes with no use.

REPLACED USED ELECTRODES

Electrode lifetime is around 40 uses. You will find refills from your usual retailer or online: www.sport-elec.co.uk.

MAINTENANCE

- ▶ After use, the electrode must be covered with its transparent operculum supplied.
- ▶ The generator must be cleaned with a dry cloth. No household product or liquid must be used to clean it.

- ▶ The Farmadolor must be stored in a dry place away from heat.
- ▶ Similarly, if you don't plan to use your Farmadolor for a long period of time: remove the batteries.
- ▶ The Farmadolor is meant for private use. For reasons of hygiene, the electrode should only be used on one person.

Warranty / Lifecycle..... 2 years

Never try to repair or modify your Farmadolor device by yourself or by an unauthorized repair service. This will void the warranty and may be dangerous. And likewise, solely use the Farmadolor with compatible electrodes manufactured by SPORT-ELEC®.

FR

EN

DE

ES

IT

NL

ENVIRONMENT

When the product is at the end of life, please ensure compliance with national rules.

We thank you for disposing of used device in an environmentally-friendly by using the existing public collection.

THE DEVICE

Software version	1
Type of current	TENS
Number of programs	1
Output voltage	max. 40 Vpp out of 1000 Ω
Pulse width/frequency	50 μs / 70 and 90 Hz 
Power	2 button cells CR2450 (supplied)
Dimensions/weight	65x59x15 mm - 24 g
Automatic stop	Yes, around 60 min.
Safety pilot	Yes
Non-contact detection	Yes
Power indicator	with 3 light indicators
Defective/low battery indicator	Yes
Storage temperature	0°C to + 50°C
Storage relative humidity	10 % to 90 %
Temperature of use	+ 5°C to + 40°C
Relative humidity of use	20 % to 65 %
Electrode connection	2 push buttons
BF type	device in contact with the  patient, excluding the heart muscles.

Hazardous substances..... RoHS

The devices are built without material that may have a harmful impact on the environment.

Ingress Protection Rating IP 22

Protected against the ingress of foreign object >12.5 mm & against dripping water when tilted up to 15°

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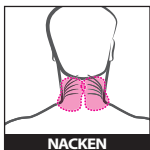
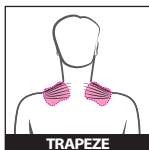
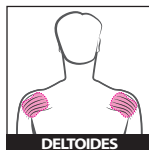
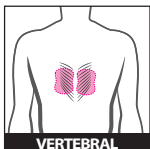
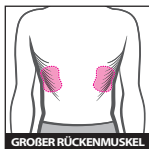
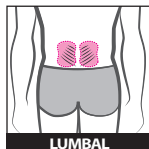
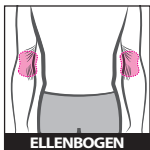
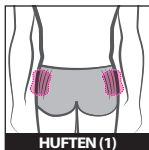
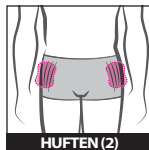
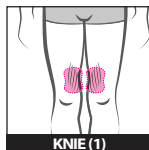
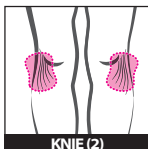
GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC EMISSIONS

The FarmaDolor is intended for use in the electromagnetic environment specified below. The customer or the user of the FarmaDolor should assure that it is used in such an environment.

EMISSIONS	COMPLIANCE	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
RF emissions CISPR 11	Group 1	The FarmaDolor uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The FarmaDolor is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC IMMUNITY

The FarmaDolor is intended for use in the electromagnetic environment specified below. The customer or the user of the FarmaDolor should assure that it is used in such an environment.

IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines and dc lines	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) and neutral ± 2 kV line(s) to earth	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % U_T (>95 % dip in U_T) for 0,5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5s	Not applicable	Mains power quality should be that of a typical commercial or hospital environment. If a dips or an interruption of mains power occurs, the current of the FarmaDolor may be dropped off from normal level, it may be necessary to use uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE - U_T is the a.c. mains voltage prior to application of the test level

GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC IMMUNITY

The FarmaDolor is intended for use in the electromagnetic environment specified below. The customer or the user of the FarmaDolor should assure that it is used in such an environment.

IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	Not applicable	<p>Portable and mobile RF communications equipment should be used no closer to any part of the FarmaDolor, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance $d = 1.2\sqrt{P}$ $d = 1.2\sqrt{P}$ 80 MHz to 800 MHz $d = 2.3\sqrt{P}$ 80 MHz to 2.5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation Distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the FarmaDolor is used exceeds the applicable RF compliance level above, the FarmaDolor should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the FarmaDolor.*
- b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.*

RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE RF COMMUNICATIONS EQUIPMENT AND THE FARMADOLOR

The FarmaDolor is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the FarmaDolor can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the FarmaDolor as recommended below, according to the maximum output power of the communications equipment.

RATED MAXIMUM OUTPUT POWER OF TRANSMITTER (W)	SEPARATION DISTANCE ACCORDING TO FREQUENCY OF TRANSMITTER (M)		
	150 KHZ TO 80 MHZ $d = 1.2 \sqrt{P}$	80 MHZ TO 800 MHZ $d = 1.2 \sqrt{P}$	800 MHZ TO 2.5 GHZ $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 - At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 - These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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