**SPORT-ELEC®**

**MULTISPORTPRO**

*MSPRO4M*

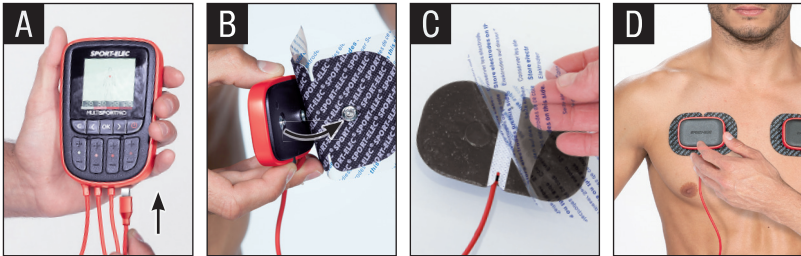
**Mode d'emploi**  
**User manual**

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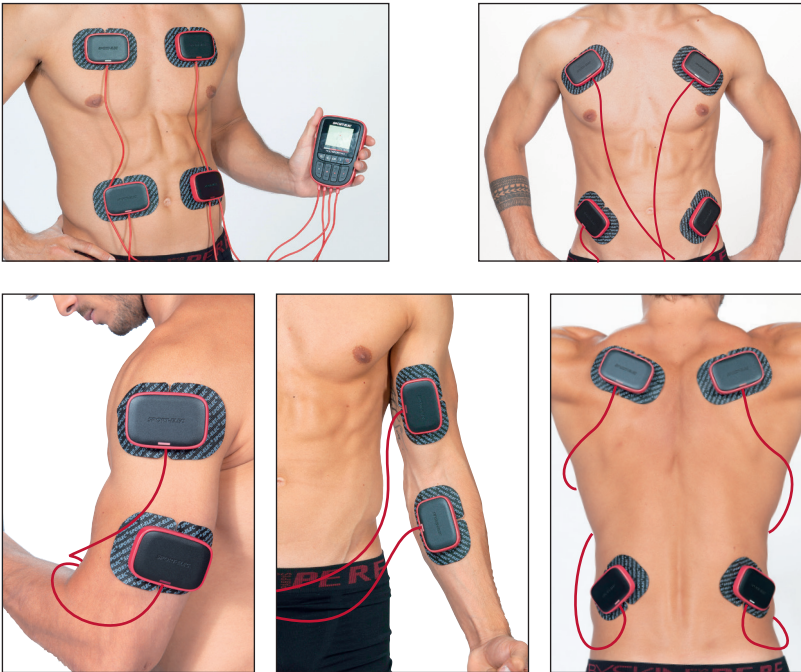
## DESCRIPTION DE L'APPAREIL

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# MULTISPORTPRO 4 MODULES (MSPRO4M)

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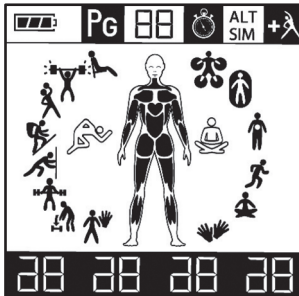
# MULTISPORTPRO 4 CHANNELS (MSPRO4M)

## OVERVIEW OF THE DEVICE (see captioned diagrams on page 3)

- 1. OK key**  
Validation after each selection.
- 2. Left key (<)**  
Back or select another choice.
- 3. C key (clear)**  
Back after validating.
- 4. Right key (>)**  
Choice selection key (programs, muscular zones).
- 5. ON/OFF key / Emergency stop (⏻)**  
Turns the unit On/Off.
- 6. 4 buttons for power regulation (+ / -)**  
+ : Increases the power / Level 1 to 10.  
- : Decreases the power / Level 10 to 1.  
Note: 11 several presses for adjusting intensity level
- 7. Connexion module**
- 8. Colour module indicator**  
Distinguishes each module for setting of the speed adjustment (+/-).
- 9. USB connection sockets**
- 10. Charging battery connection socket (DC IN jack)**  
Can be used to connect the power adapter.
- 11. Locking system slider**  
Push the button to the left to operate the device.  
Push the button to the right to charge the device.
- 12. Fastener buttons**  
Can be used to attach one dual adhesive pad equipped with snap fasteners.

## DESCRIPTION OF THE SCREEN

- Battery level
- Displays number of category programs
- Displays the pre-set program length (in minutes)
- ALT** Alternate muscles option (option available for the program categories from 02 to 07).  
Train 2 muscle groups alternately.
- SIM** Simultaneous muscles option (default choice).  
Train 2 muscle groups simultaneously.
- Enable/disable warm-up phase
- SPORT program categories
- HEALTH program categories

## BEFORE USING THIS UNIT

You have just acquired an effective and powerful professional unit. Take your time to discover how to use it.

- Read the instructions carefully.
- Before using it, charge the unit (pictures 1, 2, 3 page 3).
- Test your unit on your quadriceps using the quick test procedure below.

## NON-THERAPEUTIC DEVICE

## OPERATION QUICK TEST

### TESTING THE DEVICE ON YOUR QUADRICEPS

- Connect one electrode with snap fasteners to the first module (white).
- Remove the plastic and stick to the skin.
- Start the unit using the ⏻ key and press the OK key 6 times.
- Increase gradually the power on this channel pressing successively 11 times.
- Once this procedure is done, you can start a program that best suits your needs.

# PROGRAM CATEGORIES

## SPORT PROGRAM CATEGORIES

### 01. PREPARATION TO MUSCULAR EFFORT

Lowers the impedance of the skin barrier. Prepares your muscles for the currents emitted by the programs.

**Duration:** 2 min active phases

### 02. RESTORE MUSCLE VOLUME

Prevents and fights loss of muscle volume resulting usually from a lack of use of a limb following a period of immobilisation.

**Duration:** 40 min active phase + 6 min recovery

### 03. REINFORCE MUSCLE VOLUME

To apply only on muscles with a normal volume. The program categories 04 and 05 are used to develop muscle strength.

**Duration:** 20 min active phase + 6 min recovery

### 04. MUSCLE RESISTANCE

The stimulation frequencies aim at increasing the ability to sustain major muscle efforts.

**Duration:** 30 min active phase + 6 min recovery

### 05. MUSCLE ENDURANCE

After using the "Muscle Resistance" program category to acquire the ability to sustain major efforts, use this program category to extend high intensity muscle efforts over time by maximum stimulation of slow fibres.

**Duration:** 30 min active phase + 6 min recovery

### 06. MUSCLE FITNESS

Use program category to maintain the level of muscle endurance, resistance, and volume acquired.

**Duration:** 30 min active phase + 6 min recovery

### 07. EXPLOSIVE FORCE

This category is a logical extension of categories 03 and 04. It allows reaching maximum muscle force by stimulating very high speed fibres, with maximum contraction intensity.

**Duration:** 30 min active phase + 6 min recovery

### 08. RECOVERY

Using a frequency range between 0.5 and 8 Hz, this category ensures muscles recover faster through relaxing, endorphin-based effects, and increases blood flow.

**Duration:** 20 min active phase

## WELL BEING

### 09. MASSAGE

This program category uses frequencies modulated between 3 and 10 Hz for optimum massaging effect.

**Duration:** 20 min active phase.

### 10. MUSCLE RELAXING

The frequency range between 0.25 and 3.5 Hz of this program category accelerates muscle relaxing.

**Duration:** 20 min active phase.

### 11. ACTIVE RECOVERY

Use this program immediately after performing intense efforts.

**Duration:** 20 min active phase.

### 12. RELIEVES PAIN

This program operates by across-skin nerve stimulation, delivering a range of pulse frequencies in the local area where the electrodes are placed.

**Duration:** 20 min active phase.

### 13. MUSCLE TENSION

This program relieves muscle tensions effectively.

**Duration:** 20 min active phase.

### 14. HEAVY LEGS

This program relieves numb legs sensation.

**Duration:** 26 min active phase.

## I. CHARGING THE BATTERY

The device contains a rechargeable battery, immediately following purchase, simply charge it for 4 hours by plugging the unit into the mains power supply.

It is not possible to use the device whilst the battery is charging.

If the device remains unused, charge the battery for 4 hours (maximum) every 4 months.

The device switches itself to stand-by mode if the battery needs charging, this is to prevent the battery to be unloaded.

### TO CHARGE THE UNIT (see diagrams 1, 2, 3 page 3)

- ▶ Disconnect the 4 modules cables from the unit.
- ▶ Close the locking system by pushing the button to the right (4 USB locked, jack DC IN open). .....
- ▶ Connect the charger supplied with the unit.

**IMPORTANT! Before charging batteries, make sure that the locking system is properly closed.**

### DISPLAY DURING THE BATTERY CHARGE

- ▶ While charging the battery, the indicator battery displays , , then to show that the battery is charging.
- ▶ Unplug the AC adapter in order to block out the energy supply when the battery is fully charged.
- ▶ When unplugging the AC adapter, the screen switches off.
- ▶ When the battery indicator is empty , it means that the battery is discharged.

## II. STARTING THE DEVICE

### 1 DISPLAY - POWERING THE DEVICE ON

Beware to starting the device, press briefly maximum 1 second the red key in order to switch the device on. Wait a complete switch on of the unit before pressing another key.

All pictograms display and shut off, then all front muscles blink.

If you press permanently, the screen will blink and display all the available programs.

Press the key for switching the device off.

Or for returning to the previous menu, press the key or press briefly the key twice.

### 2 SELECTING THE MUSCLE ZONE(S) TO STIMULATE

#### ▶ Choosing the 1<sup>st</sup> zone

The screen displays , meaning you can choose a first muscle zone.

↳ To choose the front muscles, validate with the **OK** key: the 1<sup>st</sup> muscle zone at the bottom blinks: quadriceps.

To choose the back muscles, press the **>** key: the 1<sup>st</sup> muscle zone at the bottom blinks: gastrocnemius (calves)

To return to the front muscles, press the **<** key.

↳ Press the **>** key to move upwards and scroll the different muscle zones.

To go back, press the **<** key.

↳ After selecting the muscle group, validate using the **OK** key

#### ▶ Choosing the 2<sup>nd</sup> zone

All pictograms blink and the screen displays . The screen proposes to choose a 2<sup>nd</sup> muscle zone:

↳ To choose a 2<sup>nd</sup> muscle zone, validate again either the front muscles or the back muscles, validate **OK**.

Choose again another muscle zone, then validate **OK** key.

#### Recommendations

The pictograms for upper muscles and for lower muscles are different. Avoid selecting upper muscles and lower muscles, as the unit would give priority to upper muscles, reducing somewhat the yield of work on lower muscles.

### 3 SELECTING THE PROGRAM CATEGORY

Symbols of the "Sport" category programs flash while displayed (left side of the screen)

↳ If you choose the SPORT program categories , validate **OK** .

↳ To choose the HEALTH program categories , press the **>** key and validate **OK** .

Scroll through the program categories by pressing the **>** key:

↳ The screen first displays the 1<sup>st</sup> lower pictogram while displaying the corresponding category number at the top.

↳ To go back, press the **<** key.

You have now chosen the muscle zone(s) to be stimulated and the type of program.

The device varies automatically the electric pulses according to the targeted body areas

#### 4 CHOOSE THE WARM-UP PHASE AND ALT/SIM OPTION

Only for categories 02 to 07.

- ▶ The screen proposes a warming-up phase before starting the active phase of the program. The WARM-UP PHASE pictogram blinks in the top right corner. To select it, press **OK**, otherwise press the **<** key and validate **OK** (it disappears from the screen).
- ▶ Then the **ALT/SIM** pictogram blinks. To select **ALT**, press the **<** key, then validate **OK**. To select **SIM**, press the **>** key, then validate **OK**. If you have selected the warm-up phase, **SIM** is displayed when starting the program, as the warm-up phase is only in the simultaneous mode.

The pictogram for the program category chosen is lit, the muscle zone(s) chosen blink alternately.

The screen is displayed:

- the battery level,
- the phase duration (countdown in minutes),
- the timer pictogram,
- the **ALT** or **SIM** option and warm-up phase if you have selected them.

#### 5 LAUNCHING THE START - ADJUSTING INTENSITY

You must press the **+** buttons matching the cables to the pads (or to the accessories) for starting a program.

**For increasing the intensity level and changing the strength of the stimulation, press 11 times to move to the next level.**

The intensity level(s) is/are displayed at the bottom of the screen. Do not start a program if the battery level is low.

The intensity ranges vary during the session according to the program.

**Warning! The unit is equipped with a security system that locks the launch of a program if the electrode pads are not placed correctly on the skin.**

#### 6 PROGRAM PERFORMANCE

##### ▶ Use of the **C (Clear)** key

During the program, you can press the **C** key to cancel the ongoing phase.

The next phase starts unless you have reached the end of the program.

In this case, this key allows returning to selecting the 1<sup>st</sup> muscle zone (as at normal end of program).

##### ▶ Disconnecting the electrodes during the program

If an electrode is disconnected, the channel concerned is reset to **00** and blinks for 30 sec, before returning to a fixed **00**.

During these 30 sec, the electrode(s) can be reconnected, and power restored to the level required.

#### 7 END OF PROGRAM - STOPPING

At the end of the session, the screen proposes to start a new session by blinking the front muscles to select the 1<sup>st</sup> muscle zone.

Restart the starting procedure if you wish to continue on another muscle zone.

Otherwise, press the **0** key to stop.

The unit switches to standby after 5 minutes idle.

## III. USING THE UNIT

### A. USING THE DEVICE WITH THE BELT

The belt has 2 pairs of snap fasteners onto which 2 wireless electrodes specially designed by SPORT-ELEC® are clipped.

To obtain new electrodes, contact your usual retailer or order them from our website [www.sport-elec.com](http://www.sport-elec.com).

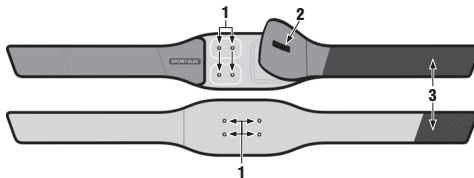
#### **Warning!**

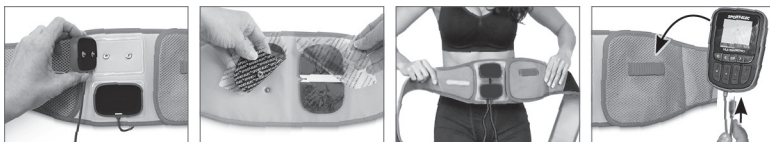
Never use the belt without attaching the 2 electrodes provided.

Only use electrodes supplied by SPORT-ELEC®.

#### PRESENTATION OF THE BELT :

- 1 = Snap fasteners
- 2 = Wire storage flap (with a lopp fixing device)
- 3 = Scratch strip

### CONNECTION AND POSITIONING OF THE BELT

- ▶ Open out the belt. Check that your device is switched off.
- ▶ Fix 2 modules onto the outside of the belt using the snap fasteners.
- ▶ Fix the electrodes onto the inside of the belt using the snap fasteners. Remove the plastic backing sheet from the electrodes.
- ▶ Place the belt around the waist and adjust it using the Velcro strip, so that there is a good contact between the electrodes and the skin.
- ▶ Once the belt is in position, make yourself comfortable and switch it on following the «Starting the device» procedure. You can carry out your normal activities while using the device.
- ▶ If you experience any tingling, switch off the device and check that the electrode is in contact with the skin. If the problem persists, contact the after-sales service: [sav@sport-elec.com](mailto:sav@sport-elec.com).

### END OF PROGRAM

- ▶ Once the program has finished, switch off the device and unplug it from the belt.
- ▶ Remove the electrodes and moisten them slightly with a few drops of water before replacing them onto their transparent plastic backing sheets.

## B. USING THE DEVICE WITH THE ELECTRODES

### GETTING STARTED

- ▶ Connect each module to the unit by matching the color indicators (photo A page 4).  
Note: You can use the device with 1, 2, 3 or 4 modules.
- ▶ Fix the electrodes to the module(s) using the snap fasteners (photo B page 4).

### POSITIONING OF THE ELECTRODES

- ▶ Remove the plastic backing sheet from the electrodes (photo C page 4).  
Moisten them by spreading a few drops of water onto the adhesive gel using your fingertips and wait 2 minutes (photo D page 4).
- ▶ Position the electrode(s) onto your body.  
Note: The electrodes are for personal use. For hygiene reasons, they must always be used by the same person.

### STARTING THE PROGRAM

Start your device by following the "Starting the device" procedure (chapter II).  
Contractions should be firm but must never cause any pain.

If the electrodes cause you any discomfort: reduce the power levels, move the electrode(s) and then gradually increase the power.  
Also ensure that the electrodes are sufficiently moistened.

Now that the settings have been optimized, let your device get to work for the full duration of the program.

### END OF PROGRAM

Once the program has finished, switch off the device by pressing the  key, then disconnect the modules.  
Gently remove the electrodes from your skin.

Moisten them slightly with a few drops of water and replace them carefully onto their transparent plastic backing sheet.  
Keep the adhesive electrodes in a plastic bag in a cool place (for example, a separate compartment in your refrigerator).

## IV. RECOMMENDATIONS OF USE

- ▶ The unit is recommended for physiotherapists for functional re-education.
- ▶ You can restart a program's cycle on another muscle zone or change the program while remaining on the same muscle zone, without exceeding however 50 min use per muscle zone and per day.
- ▶ Clean the unit and its cables with a cloth dampened with a disinfecting solution diluted in water at 1/10<sup>th</sup>.
- ▶ The adhesive electrodes can be rehydrated before and after use with a disinfecting agent.
- ▶ The adhesive electrodes must be repositioned on their support, and stored in a fresh storage between +5°C and +10°C.
- ▶ The adhesive electrodes have a lifetime of about 40 uses, you can order them directly from the online shop [www.sport-elec.com](http://www.sport-elec.com) or from your reseller.

## WARNINGS

- ▶ Keep away from children or people with mental disabilities. The device contains small pieces that can be swallowed.
- ▶ In order to prevent any risk of strangulation, please keep the electrodes wires away from children.
- ▶ The device is meant for personal use. For reasons of hygiene, the electrode should only be used on one person.
- ▶ Do not use the device by driving or by working on a machine.
- ▶ Do not use this electro stimulation unit in a damp environment, a bathroom, a sauna or close to electric machinery connected to mains and earthed, or even piping earthed.
- ▶ Use this device in a clean place (without dust nor dirt...).
- ▶ Do not, under any circumstances, attempt to open or repair the Multisportpro yourself! Only have repairs carried out by the customer service department.

## TROUBLE SHOOTING

If your device doesn't work properly, do not use it and contact your reseller.

## PRECAUTIONS OF USE

- ▶ Do not use the unit on your face.
  - ▶ Do not leave within reach of children.
- Ask for advice from your physician:**
- ▶ if you have electronic or metallic implants (pins...),
  - ▶ in case of skin condition (wounds...).

## ⚠ CONTRA-INDICATIONS

**Do not use SPORT-ELEC®:**

- ▶ On the heart area.
- ▶ If you carry a pacemaker.
- ▶ If you are pregnant.
- ▶ During first 6 weeks after baby birth.
- ▶ In case of neuromuscular diseases.
- ▶ In case of hemorrhagic diseases.
- ▶ On the face and skull.

## RECOMMENDATIONS FOR MAINTAINING AND CLEANING

Clean the unit with a tissue dampened with a disinfecting solution diluted with water at 1/10<sup>th</sup>.

# V. TECHNICAL SPECIFICATIONS

## BELT

**References :** CTBB2MEAV2

**Size (unisex) :** from 34 to 60

**Length of the unfolded belt :** 122 cm

**Fabric :** 35% sponge, 20% elastic fabric, 15% polyester, 15% nylon, 10% PVC + copper, 2% rubber, 3% non-woven fabric

## ADHESIVE ELECTRODES

**Reference :** EASF110x71

**Contact area :** 67.5 x 47 mm (x2)

**Estimated life expectancy :** 40 uses  
Class I medical device (Directive 93/42/CEE)

Adhesive, hydrophilic and hypoallergenic electrodes

Please order new compatible adhesive electrodes on [www.sport-elec.com](http://www.sport-elec.com).

## AC ADAPTOR

**Reference :** MN-A403-E120

**Input :** 100-240V, 50/60Hz, 0.25 A max

**Output :** 5V = 0.6A

The adapter is part of the equipment. Only use the AC adaptor supplied with the device.

Please replace this adaptor should it fail (contact SPORT-ELEC® or order at [www.sport-elec.com](http://www.sport-elec.com)).

**UNIT**

Software version.....	E
Type of current.....	Dual phase
Number of programs.....	88 "SPORT" programs - 6 "HEALTH" programs
Number of program variations (or phases).....	344
Number of adjustable independent outputs.....	4
Maximum intensity on a 500 Ω load/channel.....	 100 mA / 500 Ω / channel
Frequency range.....	0.25 to 160 Hz
Pulse width range.....	70 to 360 μs
Power supply.....	LiPo Rechargeable battery 3.7V - 1020 mAh
Unit dimensions and weight.....	83.5 x 125.5 x 29.61 mm - 340 g
Automatic stop at end of each program.....	Yes
Activation safety.....	100%
Temperature for stocking / Relative humidity.....	0°C to + 45°C / 10% to 90%
Temperature for use / Relative humidity.....	5-40°C / 20-65%
Caution!.....	 Read the manual thoroughly
Guarantee of the unit / Lifecycle.....	2 years (unit)

CE conformity declaration

Device in contact with the patient, excluding the heart area.

**RoHS**

These units are free of materials likely to have a negative impact on the environment.

**IP 22**Protected against the ingress of foreign  $\geq 12.5$  mm and against dripping water when tilted up to 15°.Caution:  
Read the instructions for use.Maximum intensity per channel up to 1 kΩ:  
90 mA.Relative humidity for storage:  
from 10 to 90%.Temperature for storage:  
from 0 to 40°C.

Name of the manufacturer bringing the device to the market.

Please contribute to protect the environment by observing the disposal instructions.

The manufacturer respects the environmental law and contributes to the cost of recovery and recycling.

Follow the user manual instructions.

Do not use if you carry a pacemaker.

Do not use if you are pregnant.

This device is intended for domestic use.

Double insulated electrical appliance (&gt; 50V).

**VI. GUARANTEE**

We provide a 24 months guarantee for products supplied by us, from the date of purchase.

The adhesive electrodes and the belt are for personal use. For hygiene reasons, they can therefore neither be returned nor exchanged.

The life expectancy of the adhesive electrodes is estimated at approximately 40 uses.

Never attempt to repair or modify the device yourself or by a repairman not authorized by SPORT-ELEC®. The guarantee becomes null and void if repairs have been carried out by non-authorized persons or if the replacement parts used are not from the same manufacturer. Similarly, only use compatible accessories manufactured by SPORT-ELEC®.

During the guarantee period we repair, free of charge, manufacturing defects or faulty parts, reserving the right to decide if certain parts need repairing or replacing, or if the device itself must be replaced.

Any damage resulting from improper use (knocks, plugging into a direct current, wrong voltage...) does not fall under the guarantee; normal wear and tear does not affect the proper functioning or value of the device.

The guarantee is only valid if the date of purchase and the stamp and signature of the retailer appear on the guarantee certificate or upon presentation of the till receipt (see "guarantee certificate" at the end of this manual).

No other claims fall under the guarantee, unless otherwise provided by law.

**ACCESSORIES GUARANTEE**

The electrodes and the belt are designed for personal use. For reasons of hygiene, there will be no trade-in nor exchange.

**DO NOT FORGET TO ACTIVATE YOUR WARRANTY ONLINE: [www.sport-elec.com/activation](http://www.sport-elec.com/activation)**

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Standards IEC 60601-1 - 2: 2007 on  
electromagnetic compatibility.  
For the product Multisport PRO.

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## GUIDANCE AND MANUFACTURER'S DECLARATION – ELECTROMAGNETIC EMISSIONS

The Multisport PRO is intended for use in the electromagnetic environment specified below. The customer or the user of the Multisport PRO should assure that it is used in such an environment.

EMISSIONS	COMPLIANCE	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
RF emissions CISPR 11	Group 1	The Multisport PRO uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Multisport PRO is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

## GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC IMMUNITY

The Multisport PRO is intended for use in the electromagnetic environment specified below. The customer or the user of the Multisport PRO should assure that it is used in such an environment.

IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) and neutral	±1 kV line(s) and neutral	
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % U <sub>r</sub> (>95 % dip in U <sub>r</sub> ) for 0,5 cycle	<5 % U <sub>r</sub> (>95 % dip in U <sub>r</sub> ) for 0,5 cycle	Mains power quality should be that of a typical commercial or hospital environment. If a dips or an interruption of mains power occurs, the current of the Multisport PRO may be dropped off from normal level, it may be necessary to use uninterruptible power supply or a battery
	40 % U <sub>r</sub> (60 % dip in U <sub>r</sub> ) for 5 cycles	40 % U <sub>r</sub> (60 % dip in U <sub>r</sub> ) for 5 cycles	
	70 % U <sub>r</sub> (30 % dip in U <sub>r</sub> ) for 25 cycles	70 % U <sub>r</sub> (30 % dip in U <sub>r</sub> ) for 25 cycles	
	<5 % U <sub>r</sub> (>95 % dip in U <sub>r</sub> ) for 5s	<5 % U <sub>r</sub> (>95 % dip in U <sub>r</sub> ) for 5s	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

*NOTE - U<sub>r</sub> is the a.c. mains voltage prior to application of the test level*

## GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC IMMUNITY

The Multisport PRO is intended for use in the electromagnetic environment specified below. The customer or the user of the Multisport PRO should assure that it is used in such an environment.

IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT – GUIDANCE
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF Communications equipment should be used no closer to any part of the Multisport PRO, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d=1.67\sqrt{P}$ $d=1.67\sqrt{P}$ 80 MHz to 800 MHz $d=2.333\sqrt{P}$ 800MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation Distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup> Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	

**NOTE 1** At 80 MHz and 800 MHz, the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations.

Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Multisport PRO is used exceeds the applicable RF compliance level above, the Multisport PRO should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Multisport PRO.
- b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

## RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE RF COMMUNICATIONS EQUIPMENT AND THE FREE ACTION

The Multisport PRO is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of Multisport PRO can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Multisport PRO as recommended below, according to the maximum output power of the communications equipment.

RATED MAXIMUM OUTPUT POWER OF TRANSMITTER (W)	SEPARATION DISTANCE ACCORDING TO FREQUENCY OF TRANSMITTER (M)		
	150 KHZ TO 80 MHZ $d = 1.17 \sqrt{P}$	80 MHZ TO 800 MHZ $d = 1.17 \sqrt{P}$	800 MHZ TO 2.5 GHZ $d = 2.33 \sqrt{P}$
0.01	0.117	0.117	0.233
0.1	0.369	0.369	0.738
1	1.167	1.167	2.333
10	3.690	3.690	7.377
100	11.67	11.67	23.33

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**NOTE 1** At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people

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